PLIF KAGE RSF

Ready, Set, Fuse!

PLIF KAGE RSF

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The PLIF Kage RSF is a lumbar PLIF cage, pre-filled with a synthetic bone substitute and pre-loaded on a disposable cage inserter.

The assembly is supplied sterile in a double-blister pack, ready-to-use and ready-to-fuse.

THE CAGE

The cage is made of PEEK and features 4 tantalum positioning markers (3 horizontal and 1 vertical).

The PLIF Kage RSF measures 25mm in length. It is supplied in three different angulations: 0°, 4° and 8°, each of which is available in the following lengths: 6*, 7**, 8, 9, 10, 11, 12 and 14mm.

* Only available for 0°

** Only available for 0° and 4°

THE BONE SUBSTITUTE

The bone substitute is an entirely synthetic phosphocalcic ceramic composed of HA (60%) and TCP (40%), designed to fit the inside of the PLIF Kage precisely.

The substitute's macroporosity enables patient bone recolonization.

THE CAGE INSERTER

The cage inserter is made of stainless steel and, providing the internal screw is removed, it can also be used as an impactor at the end of the procedure.

The cage inserter is designed for easy connection to the reusable RSF handle.

FEATURES AND BENEFITS

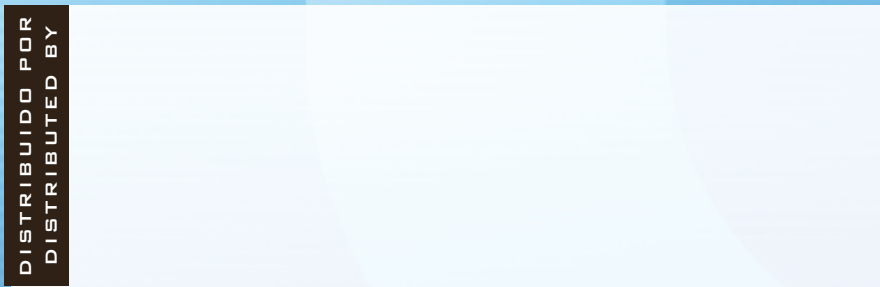
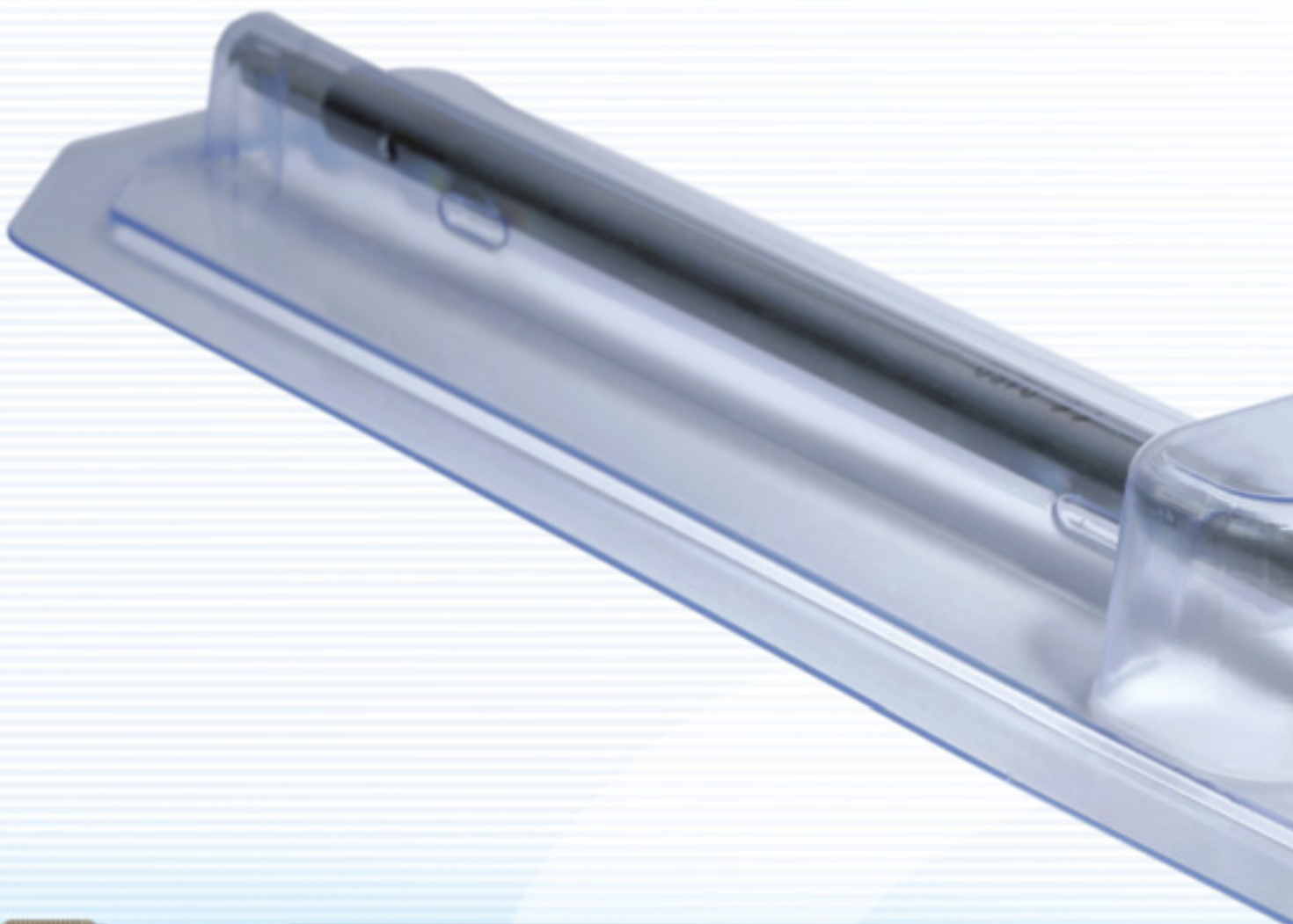
- Time saving during surgical procedure.
- Easy to use.
- No risk of defective Kage/substitute assembly.
- No risk of defective Kage/Kage inserter assembly.
- Reduced risk of cross contamination.
- Simplification of stock management in operating theatre.
- Less need for ancillary insertion instruments.
- Easy to connect with the reusable RSF handle.

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Design photos: e-montaux.fr / ref 927 Ad 01 - © KASIOS® - octobre 2010